

# Review sessions

## aim

To play back recorded sessions. If your license includes the Annotation Module you can also play back the scored markers and events.

## prerequisites

You either have rights to review the session (Technician, Administrator), you created the session (Trainer), you have received ownership or annotator rights or you are invited to view the session.


## procedure

1. Open the **Sessions** tab for a list of all the sessions.

 Image not found or type unknown

2. Double-click a row to open a session.

3. Click the play button to play it back.

 Image not found or type unknown

## To review an event

If you want to play back a specific event, set a pre-roll interval to start playing a few seconds prior to the event.


1. Click the menu button, choose **Settings** and then **Playback settings**.
2. Set the duration of the pre-roll interval and click **OK**.

By default the pre-roll interval is set to 5 seconds. The maximum duration you can set is 15 seconds. If you do not want to have a pre-roll, set the value to 0.


## To review a composite session

If your session consists of multiple locations, use the carousel control to view the different locations.

See [To open locations with the carousel control](#)

 Image not found or type unknown

The Timeline shows when a location change takes place.

 Image not found or type unknown

Use the carousel to view the video in the next location, for instance when following a participant from one room to another.

## To change the video layout

Optionally, change the video layout in the same way as with recording sessions.

See [Video layout](#)

## To change the audio

Optionally, change the playback audio in the same way as with recording sessions.


See [Change Audio source and volume in preview](#)

# To review the last time fragment

Click the quick review button to let the video jump back and replay the episode. Click it multiple times to replay a longer episode.

 Image not found or type unknown

By default, the video jumps back five seconds. To select another time interval, click the **Menu** button at the top of the Viso window and choose **Settings > Playback settings**. Next, specify the interval. You can also control here if you want to pause after quick review or continue the playback.

 Image not found or type unknown


# Session sorting options

Click one of the column headers to sort the sessions, by name, date, location, the person that owns the session or the person that has been assigned as the annotator.

To group sessions, for example by the user that created them, drag that column header (Created by in this example) to the top of the table.

 Image not found or type unknown

Unfold a group to see the individual sessions. Click the **red cross** to remove grouping.

 Image not found or type unknown

You can drag multiple column headers to the top of the table to make groups and subgroups of sessions.