

Select multiple locations to record a composite session

aim

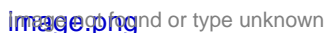
To select multiple locations for simultaneous recording. All recordings are combined in one session.

prerequisites

- The Technician account enabled multiple-location recording.
- Your user role allows session recording (Technician, Administrator, or Trainer).

procedure

1. In the **Locations** tab, select the locations.
2. Click the **Composite session** button on the toolbar.

Image not found or type unknown

A preview of the cameras in the location that you selected first appears. The toolbar displays the carousel control.

See [The carousel control](#) and [To open locations with the carousel control](#) control to set up the locations

See the third chapter of this Reference ([Prepare session recording](#)) for the procedure. Follow the procedure for each location.

NOTE The Composite Session button is enabled once you selected multiple locations.

NOTE Composite session recording is supported with a maximum number of eight locations with two cameras each.

Revision #5

Created 15 January 2024 11:38:47

Updated 31 May 2024 13:56:53 by Niek Wilmink