

Select one location and record a session

aim


To select one location for a single session recording.

prerequisite


Your user role allows session recording (Technician, Administrator, or Trainer).

procedure

1. In the **Locations** tab, double-click the **name** of the location in which you want to start recording.

 Image not found or type unknown

2. A preview of the cameras in the location appears. The toolbar displays the location name.

 Image not found or type unknown

3. Continue with the third chapter of the Reference ([Prepare session recording](#))

NOTE The location is locked for recording by others at the moment you select it in the Locations list.

NOTE You can also record a single session with the Viso Web App, see [The Viso Web App](#)

Revision #4

Created 15 January 2024 11:38:47

Updated 12 March 2024 09:16:03